 Single Patient Use

Rx ONLY

REF

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133319-01

WARNING:

In rare cases, skin may become irritated where it rubs against the headgear. If you experience a rash or discomfort, discontinue use and contact your physician.

Assembly Instructions:**Step 1: Fit the Headgear**


Fit the headgear on your head as shown in Figure 1.

Adjust the two Velcro® straps on the front of the headgear until a comfortable fit is achieved. Using a mirror can make this process easier (Figure 1).

Step 2: Attach the ADAM Circuit

Slip the headgear from your head without releasing the Velcro straps.

Select the largest size nasal pillow that is comfortable and seals at the nares securely. Insert the nasal pillows into the shell located on the end of the circuit (Figure 2). Insert the pillows only as far as the first ring on the pillows. (See inset, Figure 2.)

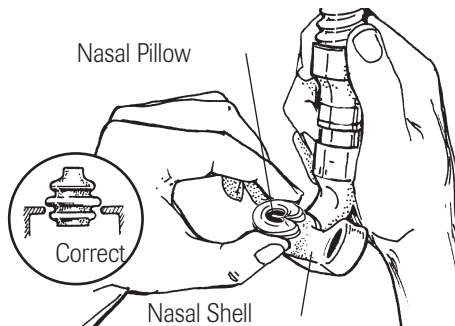**Figure 1****Figure 2**

Open the two loops on the top of the headgear and insert the ADAM circuit (Figure 3). Thread the Velcro straps back through the buckles on each loop, pressing down to secure in place. Leave the loops loose enough for adjustment before pulling snug.

Step 3: Attach the Shell Strap (Figure 4)

Center the shell strap on the nasal shell velcro pad. Position the two nasal pillows in your nares, rotating as necessary for a good seal and comfortable fit. Place two of the Velcro patches on each side of the headgear just in front of the ears. This may be adjusted later.

Position the end of each strap over the Velcro patches on the side of the headgear. Slide the circuit tubing back through the two loops on top of the headgear and retighten to hold the tubing snugly in place.

Step 4: Removing the Headgear and Circuit After Use

Pull the pillows and shell gently away from your nose and slide the headgear from your head. DO NOT disassemble the headgear. Remove the ADAM circuit from the headgear for regular cleaning.

If you experience problems with the headgear it may also be modified by recutting and sewing for a custom fit.

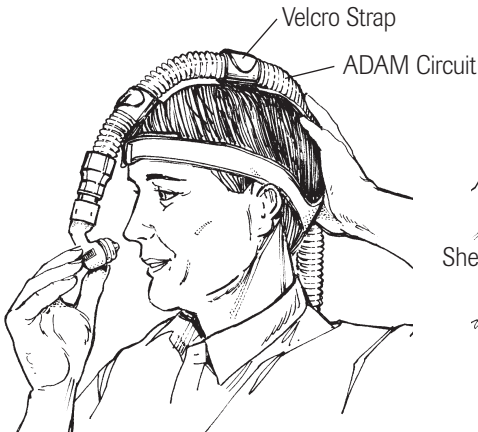
Figure 3

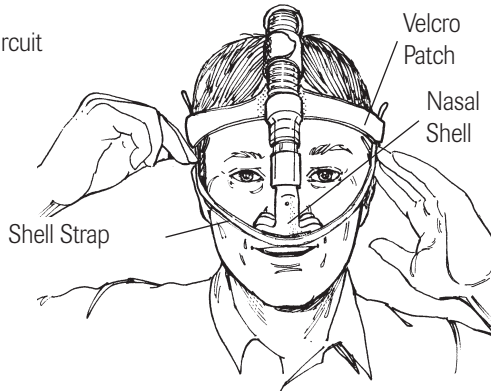
Figure 4

Step 5: Attaching the Chin Strap Accessory (Figure 5)

The chin strap is best attached with the ADAM circuit removed from the headgear.

The chin strap can be used to help prevent mouth breathing and to provide a secure fit when the patient experiences slipping of the ADAM headgear.

Position the hole in the center of the chin strap over the chin.

Pull both ends of the chin strap back across your head. Place the strap end with the Velcro strip, UP on the head just behind the top tubing loop. Attach the remaining strap to the velcro strip. Some adjustment may be required for a comfortable fit.

Slip the headgear from your head and reattach the ADAM circuit.

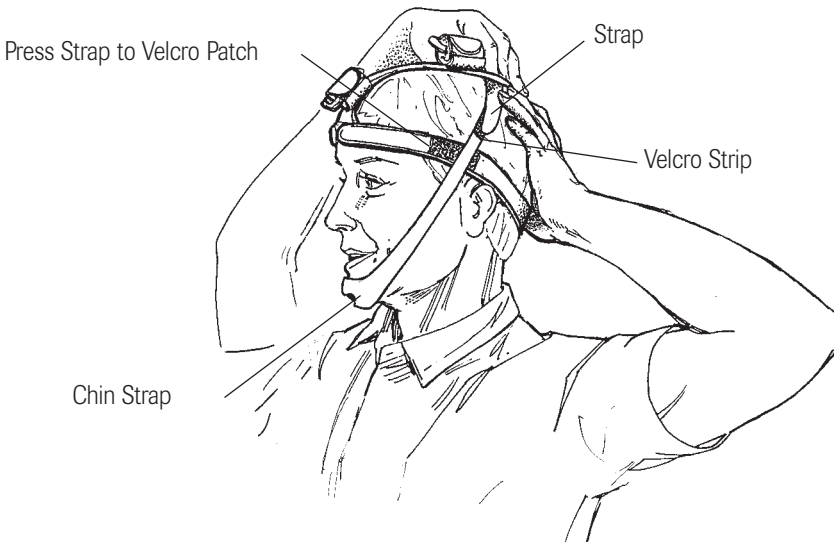
Figure 5

Cleaning Instructions:

1. Hand wash headgear in mild detergent. Do not bleach.
2. Rinse thoroughly.
3. Hang to air dry.

Replacement Velcro Patches (1 each, black)

Replacement Velcro Patches (1 each, white)

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SnugFit is a trademark of Nellcor Puritan Bennett Inc.
©2003 Nellcor Puritan Bennett Inc. All rights reserved.
**Puritan-Bennett Corporation
Tyco Healthcare UK LTD